Aladdin

LAMPADA SBIANCANTE WIRELESS
WIRELESS BLEACHING LAMP
KABELLOSE BLEICHLAMPE

DI

DENTAL INSTRUMENT DESIGN

Aladdin

LAMPADA SBIANCANTE WIRELESS

WIRELESS BLEACHING LAMP

KABELLOSE BLEICHLAMPE

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Aladdin bleaching lamp is a technology concentrate: the 3 bright intensity levels got by the L.E.D.s, allow a light powerful but cold and a revolutionary control system that through an innovative processor it let to adapt the Aladdin lamp to any patient need. Aladdin lamp can work continuously cordless without charge need for at least 3 hours.

By means of a practical and ergonomic support system the lamp can be place everywhere, the lamp terminal system can be correctly adapted to any mouth and allow the usage of many bleaching products type.

Aladdin is delivered with an introduction kit, enough for 4/6 bleaching treatments.

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The system does not need connections to the power outlet to work, because it is powered by a rechargeable battery pack.

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Through a digital panel it is possible to adjust the light intensity. Left, Centre & Right LEDs have 3 grades of light intensity (High, Normal & Low). Left, Centre & Right LEDs can be adjust their light intensity. The light intensity can be controlled by pressing L, C or R buttons.

There are three operating time modes of 5, 10 and 15 minutes which are selectable.

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Aladdin use 3 high power LEDs assuring for all the treatment time the max programmed light through an easy and intuitive control panel.

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Aladdin allows to adjust the light power that can be applied differentiating for any patient mouth quadrant.

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NO LAMP HEATING AND MAX PATIENT SAFETY

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Max high power LEDs performances, without need of a fan to cool down the lamp, assure long Aladdin life.

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Easy to place anywhere through an elegant cart wheels equipped and a balanced arm to place the head in any place. The head can be rotated within 270 degree.

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BATTERY PACK WITHOUT MEMORY, HIGH DURABILITY

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Battery pack is easy to charge through a dedicated charge system to assure an extended working operation (3 hours).

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Dimensioni Testa
Head Dimensions
Maße (Kopf): 250 - 170 - 70 mm.

Dimensioni Battery pack
Battery pack Dimensions
Maße (Battery pack): 105 - 160 - 70 mm.

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Battery Pack e adattatore
Battery Pack and charger
Battery Pack und Adapter: 1640 g.

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Aladdin is provided with a high efficiency introduction kit, made up of a Photo-Polymerizable dam, syringes of a treatment product with 38% of hydrogen peroxide and a home usage maintenance treatment product. The maintenance product has to be applied once or twice for 10 minutes assuring a durable and visible bleaching.

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